

# BIOCREDIT COVID-19 Ag


## SARS-CoV 2 Antigen test

- » The first commercialization Black Gold Particle technology In the world (high sensitivity and specificity)
- » User friendly Dual Color System (Control line: Red, Test line: Black)
- » Rapid Test Time (5~8 Min)
- » Simple detection of SARS-CoV2 antigen

### Core Technology Black Gold Particle




Differentiated Black Gold Conjugate Technology

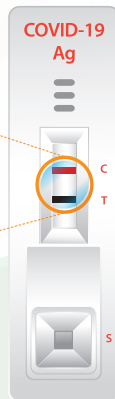

Dual Color System

8 minutes is ok!

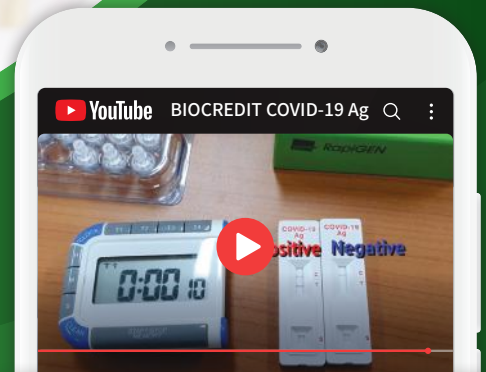

YouTube BIOCREDIT COVID-19



## BIOCREDIT COVID-19 Ag rapid test kit

<b>Intended Use</b>	Detection of SARS-CoV2 antigen 20
<b>Package</b>	Tests / kit
<b>Storage</b>	1~40°C
<b>Specimen Type</b>	Nasopharynx/ Nasopharyngeal
<b>Shelf life</b>	12 months from manufacture
<b>Time to result</b>	date 5~8 minute

# COVID-19 Rapid Diagnostic Test



## PROCEDURE


Insert the swab specimen and swirl the swab 5-10 times.

Remove the swab while gently squeezing the head of the swab.




Close the assay diluent tube with a filter cap securely.

Invert the assay diluent tube and gently squeeze it to draw 3-4 drops (90-150µl) into a sample well on the device.

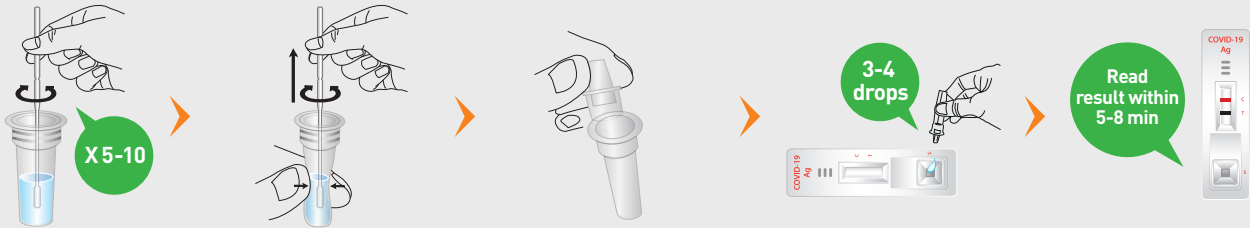Read the result within 5-8 minutes.



## INTERPRETATION

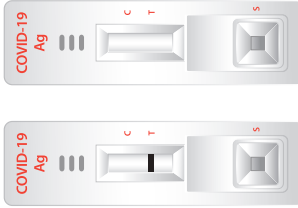NEGATIVE	POSITIVE	INVALID
		
<p>One <b>red</b> line "C" within the result window.</p>	<p>Two bands, <b>black</b> To test line and <b>red</b> "C" control line within the result window.</p>	<p>No "C" line within the result window. It is recommended that the specimen be retested.</p>



Distributed in Australia by:

 **OptiMed** TECHNOLOGIES  
OptiMed Technologies Pty Ltd  
Unit 136, 14 Loyalty Road, North Rocks NSW 2151  
T. 02 9052 7888E. info@optimedtech.com  
www.optimedtechnologies.com

 **RapiGEN** RapiGEN INC.